BLACKROLL®

BOOSTER 2.0

Instruction Manual

EN | DE | FR | NL | IT | PL | CN

Hi!

Congratulations on your BLACKROLL® BOOSTER 2.0.

This device supports your recovery and muscle relaxation through compressed vibration massage. Please note the following instructions for safe and optimal use.


Safety Instructions

- + Do not use on sensitive areas or near organs.
- + In case of pain or other contraindications, stop use immediately and consult a doctor.
- + Persons with reduced physical, sensory or mental capabilities and any person under the age of 18 must be supervised during use.
- + Do not insert electrically conductive objects into the openings of the appliance.
- + Do not insert the appliance into any orifices.
- + Never block the vibration of the device. The device must be able to vibrate freely.
- + Do not use as a support or substitute for medical applications without consulting a doctor.
- + For all applications in the area of the head, neck and upper back, use the vibration carefully and selectively.
- + Do not immerse the device in water.
- + Keep liquids away from the ventilation slots and the control buttons.
- + Keep the unit away from heat sources, children and pets.
- + Protect the unit from moisture, dust, aggressive liquids and vapours.
- + If moisture enters the unit, switch off the unit immediately and, if necessary, disconnect the mains plug from the socket.
- + Do not remove any screws or attempt to disassemble the unit.
- + Do not pass the unit on to third parties without the operating instructions.
- + In case of misuse, all warranty claims are void.
- + Do not charge or use at temperatures below 0 °C.
- + If not used for a longer period, recharge at least every 6 months and maintain a charge level above 50%.

Contraindications

Basically, there are hard indications that exclude the use of the device as well as contraindications for which an individual decision must be made as to whether the device can or may be used. If one or more of these points apply to you, please consult your doctor first:

- + Redness
- + Swelling
- + Acute pain
- + Osteoporosis
- + Damage to intervertebral discs
- + Taking anticoagulant (blood thinning) medication
- + Open, inflamed or infected skin surfaces
- + Fresh fractures
- + Increased bleeding tendency, tendency to haematoma formation (formation of bruises)
- + Tendency to embolism
- + Cardiac pacemakers (HSM, PM)
- + Direct bone or eye contact
- + Fibromyalgia
- + Rheumatic diseases
- + Joint replacement
- + Tumour diseases
- + Pregnancy
- + Diabetes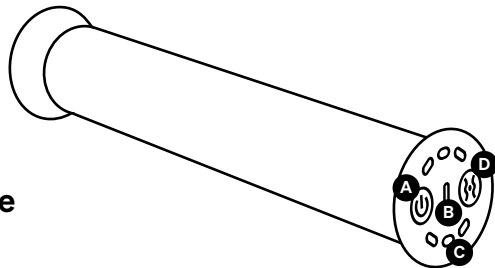

Instructions for use

- A** On/off button
- B** USB-C port
- C** Indicator for frequencies & battery level*
- D** Change frequencies

Before Use

1. Check all parts for damage.
2. Charge the battery: Charge the battery via the USB-C port using the included cable before first use. For safety reasons, the device must not be used while charging. Charging time approx. 4h. Please use a 5V 2A charger and avoid fast chargers. Steady light of all 6 LEDs = battery fully charged. Each solid LED represents approx. 15% battery level.

Startup


1. To switch the device on or off, press and hold **A** for 1.5 seconds.
2. After switching on, the LEDs of the status indicator **C** blink five times to show the battery level. Each blinking LED represents 15%.
3. While switched on, you can adjust the vibration frequency by briefly pressing **D**. There are 6 frequency levels in total.
4. Memory function: By default, the device starts with the last used setting.

Vibration frequency 15-50 Hz

The unit covers a wide and precisely controllable vibration spectrum. The frequency range is 15-50 Hz. The deep-acting vibration can irritate the pain receptors. This can lead to a reduction in the sensation of pain. The blood circulation of the muscles can be stimulated, which can promote regeneration.

When do I use which frequency?

We recommend using the low frequencies up to 30 Hz for regeneration and release of muscle tension, e.g. after training. The frequencies above 30 Hz are particularly suitable for activating the muscles, e.g. before training.

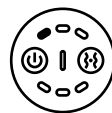approx. 15 Hz
900 RPM

approx. 20 Hz
1200 RPM

approx. 25 Hz
1500 RPM

approx. 30 Hz
1800 RPM

approx. 40 Hz
2400 RPM

approx. 50 Hz
3000 RPM

Frequency range BOOSTER: 15-50 Hz
The LED lights indicate the intensity of the vibration.

Areas of application

The BLACKROLL® BOOSTER 2.0 can be used in a variety of ways and therefore offers extensive application possibilities

1. Use inside a fascia roll

The BOOSTER 2.0 can be combined with any 30 cm BLACKROLL® fascia roll. To do so, unscrew the cap with the logo from the BOOSTER 2.0 and insert the BOOSTER into the roll. Then screw the cap firmly back on the opposite side.

Use with the BLACKROLL® TWIN or STANDARD fascia roll is particularly recommended. Set the BOOSTER 2.0 to the desired frequency and start your self-massage.

2. Use without a fascia roll as a stand-alone device

The BOOSTER vibration core can also be used independently of the fascia roll – for example as a vibrating element during training.

It is particularly suitable for active applications in which the vibration is used during movements or isometric holds – such as to activate deep muscles or for mobility exercises.

Thanks to its compact form, the BOOSTER can be easily integrated into training – for example for controlled shoulder movements, stabilizing core exercises, or holistic activation before workouts.

Exercises with the BOOSTER 2.0

Download the app now for free and start.

Cleaning

- + Do not clean the unit while it is charging.
- + Do not use water to clean the unit, only a damp cloth.
- + Do not use corrosive cleaning agents or solvents.

Technical specifications

Battery:	Rechargeable lithium-ion battery, cells: 18650
Energy (Wh):	22.2
Battery capacity (mAh):	2000
Max charging current (A):	2
Max charging voltage (V):	5
Battery replaceable?:	No (integrated module)
Charging port type:	USB-C
Rated voltage:	DC 11.1 V (3S1P 3.7V x 3)
Charging cable included?:	Yes
Power adapter included?:	No, use 5V 2A standard adapter, do not use fast charger
Protection class:	IP20
Vibration frequencies:	15–50 Hz (6 levels)
Motor speed (per level):	900 / 1200 / 1500 / 1800 / 2400 / 3000 rpm
Weight:	678 g
Standby time:	Theoretically ≥ 6 months
Charging time:	Approx. 4 hours
Operating time:	Approx. 1.5 hours (no load)
Noise level:	< 65 dB
Operating temperature:	0 °C to +35 °C

Note: During charging, the LED bar shows the current battery status. After charging is complete, the device remains off. All buttons are disabled during charging and the device cannot be used.

Disposal

The unit contains electronic components and a rechargeable battery. It must not be disposed of in normal household waste. All electrical and electronic equipment must be disposed of at government designated sites.

Disclaimer

The unit is used for exercise at your own risk. It is recommended that you seek advice from a physiotherapist, manual therapist, masseur or medical specialist if you experience severe pain before or after self-massage or recurrent injuries.

Warranty

The seller grants the buyer, if he is a consumer, the statutory warranty in accordance with the legal requirements for a period of 2 years from the purchase of the item. The Seller's warranty obligations shall initially be limited to supplementary performance, whereby the Seller shall be free to choose whether to deliver a defect-free item or to repair the defective item. If subsequent performance is unsuccessful, the buyer shall be entitled to the further statutory claims. The seller rejects warranty claims for damage caused by accident, obviously incorrect use, misuse or improper transport; which occurred as a result of repairs carried out by unauthorised personnel; which occurred as a result of use and handling not in accordance with the operating instructions; which occurred as a result of improper handling of the purchased item. Furthermore, claims for damages by the purchaser are excluded. Excluded from this are claims for damages by the buyer arising from injury to life, limb or health as well as liability for other damages based on an intentional or grossly negligent breach of duty by the seller, his legal representatives or vicarious agents. Also unaffected is the liability for breaches of duty, the fulfilment of which makes the proper execution of the contract possible in the first place and on the observance of which the buyer may regularly rely. In the event of a slightly negligent breach of such contractual obligations, the Seller shall only be liable for the foreseeable damage typical for the contract, unless it is a matter of claims for damages by the Customer arising from injury to life, limb or health.

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BLACKROLL AG
WSVK Oederan GmbH