BLACKROLL®

COMPRESSION BOOTS

Instruction Manual

EN | DE | FR | NL | IT | PL | CHN

Hi!

Congratulations on your new BLACKROLL® COMPRESSION BOOTS. Training and recovery are key to your well-being and performance. The BLACKROLL® COMPRESSION BOOTS are a battery-operated electric massage device that delivers promising results for the treatment of muscle soreness and pain. The electronically controlled pump produces different inflation and deflation patterns for an effective compression massage. Before using the COMPRESSION BOOTS for the first time, read the Instruction Manual to ensure correct use of the product.

Learn more about your BLACKROLL® COMPRESSION BOOTS.

Safety instructions

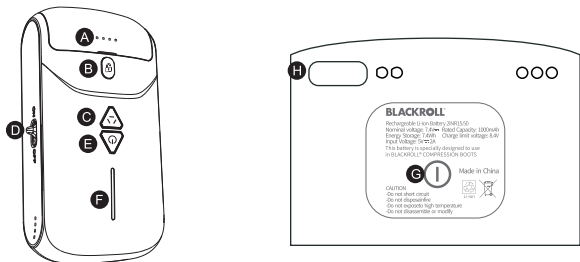
- Use product on legs only.
- Do not use with open wounds on legs or feet.
- Do not cover ventilation slots on control unit.
- Do not keep sharp objects near the product.
- Do not use while sleeping.
- Do not use near microwaves or other high-frequency devices.
- In case of pain or other contraindications, stop use immediately and consult a doctor.
- Persons with reduced physical, sensory or mental capabilities and any person under the age of 18 must be supervised during use.
- Do not insert electrically conductive objects into the openings of the device.
- Do not use as an aid or substitute for medical treatment without consulting a doctor.
- Do not immerse the appliance in water.
- Keep liquids away from the ventilation slots and the control buttons.
- Keep the appliance away from heat sources, children and pets.
- Protect the unit from moisture, dust, aggressive liquids and vapours.
- If moisture enters the unit, switch it off immediately and, if necessary, unplug it from the mains.
- Do not remove any screws or attempt to disassemble the unit.
- Do not pass the unit on to third parties without the operating instructions.
- Any misuse will invalidate the warranty.

Contraindications

Essentially there are hard indications that exclude the use of the device and contraindications in which a decision has to be made in each individual case whether the device should be used or not. Should you have one or more of the conditions listed below, please consult your doctor before using the device:

- Redness
- Swelling
- Acute pain
- Osteoporosis
- Taking anticoagulant (blood thinning) medication
- Open, inflamed or infected skin surfaces
- Fresh fractures
- Increased bleeding tendency, tendency to haematoma formation (formation of bruises)
- Tendency to embolism
- Pacemakers (HSM, PM)
- Fibromyalgia
- Rheumatic diseases
- Joint replacement
- Tumour diseases
- Pregnancy
- Acute pulmonary oedema
- Acute thrombophlebitis
- Acute infections
- Acute congestive heart failure
- Deep vein thrombosis
- Episodes of pulmonary embolism

Instructions for use

- A** Battery status display
- B** Changing the battery
- C** Massage mode
- D** On/off slider
- E** Power button
- F** Pressure indicator
- G** Charging status
- H** USB-C connection

1. Before operation

- a. Check all parts for damage.
- b. The BLACKROLL® COMPRESSION BOOTS consist of a left and a right boot. Both boot have their own control unit. Fully charge both batteries before using them for the first time. To do this, you can detach the batteries from the control unit by pressing the B button (change battery). There is a USB-C port on the

bottom of the batteries that you can connect with the supplied charging cable. Use a conventional USB 5V-2A power supply to connect the batteries to the mains and charge them (charging time: approx. 1.5 h). If use a power supply less than 2A, the charging time will be longer. The battery is fully charged when all 4 LEDs on the battery light up continuously.

The battery must be recharged when the first LED flashes during operation. If you want to check the battery status in between, you can do so by holding down the G key (charge status). Each lit LED corresponds to 25% battery charge.

2. Start operation

- a. Open the zips and put both boots on the legs one after the other. Caution: During operation, all zips must always be fully closed. Find a comfortable position for use, either sitting or lying down. To check the function, the boots can be inflated once without the legs in them. Afterwards, please always use them with the legs. Empty inflations reduce the life span of the boots and the pump.
- b. Starting the control unit: To start one of the control units, move the on/off slider (D) on the side of a unit to the on position. When the unit is on, the battery status indicator (A) and power button (E) will light up. Perform this procedure on both control units one after the other. The two units will now pair automatically. Afterwards, operation can be carried out via one of the control units. You can recognise the control unit that can be used for synchronous operation of both units after pairing by a slowly flashing power button.
- c. Establish pairing manually: If the two units do not find each other automatically, you can pair them manually. To do this, hold down the power button (E) on a control unit for at least 5 seconds. You will recognise the start of the pairing process when the pressure display starts to run up and down. As soon as this state has been established on one unit, repeat the process on the second. It does not matter which side you start with. When the pairing is complete, the pressure indicators will light up normally again. The control unit that can be used to operate both units simultaneously after pairing is indicated by a slowly flashing power button (E).

- d. **Setting the massage mode:** The massage mode determines the order in which the five air chambers are filled and how long the pressure is maintained in them. The massage mode button (C) allows you to choose between three different modes. Each mode has a different colour, which is displayed via the power button (E). For more information on the modes, see "Massage modes".
- e. **Start or stop massage:** Press the power button (E) briefly to start the compression massage. To stop the massage, press and hold the power button (E) for 2 seconds or move the on/off slider (D) to the off position.
- f. **Adjusting the pressure intensity:** You have the possibility to use five different pressure levels. The intensity is measured in the medical pressure unit mmHg. Each bar on the pressure display corresponds to 20mmHg (40/60/80/100/120mmHg). To increase or decrease the intensity, you can use the power button (E) during operation. Press it briefly to adjust the intensity.
- g. **In case of emergency or to end your session:** Setting the On/Off slider (D) on the side to Off will release pressure immediately. Use this function in emergency situations or when you want to store the boots again.

General note: If you change the pressure intensity or massage mode during operation, it may take up to 60 seconds for the display of both control units to match.

Application

The BLACKROLL® COMPRESSION BOOTS can be used to temporarily relieve sore muscles and temporarily increase circulation in the legs and feet. The application is possible both before and after sports or for general recovery of tired legs. To

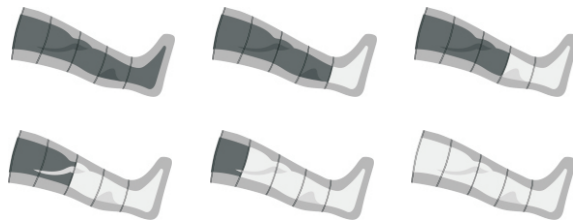
activate the muscles, we recommend sessions of 10-30min. For regeneration (e.g. after sport) you should use the Boots for at least 30 minutes. The maximum duration of use depends on your personal feeling.

Massage modes


The three massage modes are programmed for a standard time of 30 minutes. If you want to extend the application time, press the power button (E) again after 30 minutes. Before the start of each massage mode, a pre-inflation cycle of both boots is performed. The boots will adjust to your leg diameter.

The different massage modes work similar to a lymphatic drainage by a professional physiotherapist. The five air chambers create a compression with subsequent relaxation in specific sequences, always starting from the feet towards the centre of the body. After reaching the uppermost chambers and thus completing a cycle, there is a short rest phase. The cycle then starts again from the beginning. This process is repeated until the standard time of 30 minutes is reached.


Blue massage mode

Green massage mode

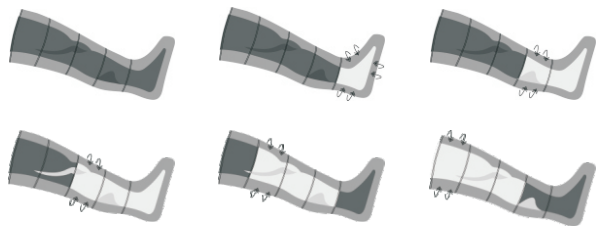Yellow massage mode

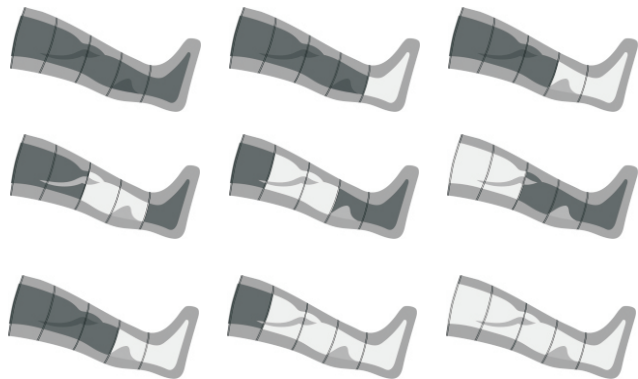You can find more information about modes and applications online on our website www.blackroll.com or in our free app.

Cleaning

Do not clean the battery while it is charging. Do not use water to clean the device, only a damp cloth. Do not use corrosive cleaning agents or solvents. Textile parts are not suitable for washing in a washing machine.

Technical specifications

Battery:	Rechargeable and removable 1000mAh lithium-ion battery.
Charging:	USB-C charging socket
Rated voltage:	DC7.4V
Rated current:	25W
Pressure:	40-120mmHg
Net weight:	2.5kg (per pair)
Standby :	60min (at medium pressure level)
Battery charging time:	1.5h
Noise level:	≤55dB

Disclaimer

The unit is used for exercise at your own risk. It is recommended that you seek advice from a physiotherapist, manual therapist, masseur or medical specialist if you experience severe pain before or after self-massage or recurrent injuries.

Disposal

The unit contains electronic components and a battery. It must not be disposed of in normal household waste. All electrical and electronic equipment must be disposed of at government designated sites.

Warranty

The seller grants the buyer, if he is a consumer, the statutory warranty in accordance with the legal requirements for a period of 2 years from the purchase of the item. The Seller's warranty obligations shall initially be limited to subsequent performance, whereby the Seller shall be free to choose whether to deliver a defect-free item or to repair the defective item. If subsequent performance is unsuccessful, the buyer shall be entitled to the further statutory claims. The seller rejects warranty claims for damage caused by accident, obviously incorrect use, misuse or improper transport; which occurred as a result of repairs carried out by unauthorised personnel; which occurred as a result of use and handling not in accordance with the operating instructions; which occurred as a result of improper

handling of the purchased item. Furthermore, claims for damages by the purchaser are excluded. Excluded from this are claims for damages by the buyer arising from injury to life, limb or health as well as liability for other damages based on an intentional or grossly negligent breach of duty by the seller, his legal representatives or vicarious agents. Also unaffected is the liability for breaches of duty, the fulfilment of which makes the proper execution of the contract possible in the first place and on the observance of which the buyer may regularly rely. In the event of a slightly negligent breach of such contractual obligations, the Seller shall only be liable for the foreseeable damage typical for the contract, unless it is a matter of claims for damages by the Customer arising from injury to life, limb or health.

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BLACKROLL AG
WSVK Oederan GmbH